

ENDO-ALERT

ENDOSCOPE TAG

INTENDED USE INFORMATION:

SurgiHealth Solutions Endo-Alert Endoscope Tags are designed to easily communicate post-procedure and post cleaning time information.

APPLICATION (1 Hour):

1. Fill in the information fields on the tag.
2. Press the button to activate the timer then attach the tag to the soiled endoscope before packing for transport.
3. Once the timer bar is completely filled endoscope reprocessing should begin.

APPLICATION (7 Days):

1. Fill in the information fields on the tag.
2. Press the button to activate the timer then attach the tag to the clean endoscope before storing.
3. Once the timer bar is completely filled, the endoscope is ready to be reprocessed.

STORAGE:

Tags should be stored in a dry area and where cross-contamination can be prevented. Do not store tags in direct sunlight.

DISPOSAL:

There are no special disposal instructions for this item.

ISSUES:

There are no known product issues or warnings.

PRODUCT NUMBERS SUPPORTED:

SH-SCT-1H	SH-SCT-7H	
-----------	-----------	--

ADDITIONAL INFORMATION:

This is a single use product.



Manufactured For: SURGIHEALTH SOLUTIONS
2122 SCHAPPELLE LN, CINCINNATI, OH 45240
PHONE: 844.88.SURGI | WEB: SURGIHEALTH.COM