

EZ-TEAR

TRAY FEET PROTECTORS

INTENDED USE INFORMATION:

SurgiHealth Solutions EZ-TEAR™ Tray Feet Protectors are specifically designed to avoid tearing or puncturing surgical tray wraps, an issue often caused by the feet and edges of surgical trays.

APPLICATION:

1. Along the perforated lines, bend the EZ-TEAR™ Tray Feet Protectors to create different sizes based on your needs.
2. Prior to wrapping, position the EZ-TEAR™ Tray Feet Protectors at the base of the tray, ensuring they cover the tray's feet and/or edge.

QUALITY INSPECTION:

Ensure that EZ-TEAR™ Tray Feet Protectors are free from tears, debris or contaminants prior to use.

STORAGE:

EZ-TEAR™ Tray Feet Protectors should be stored in a dry area where cross-contamination can be prevented. Do not store in direct sunlight.

DISPOSAL:

There are no special disposal instructions for this item. Material can be recycled.

ISSUES:

There are no known product issues or warnings.

STERILIZATION:

Cycle Type	Cycle Temperature	Minimum Cycle Time
Pre Vacuum	134° C (273° F)	3 minutes
Pre Vacuum	132° C (270° F)	4 minutes
Immediate Use Pre Vacuum	132° C (270° F)	3 minutes
Gravity	121° C (250° F)	30 minutes
STERRAD® 100S, 200, 50, NX, 100NX	All Cycles	All Cycles
V-PRO®	All Cycles	
STERIZONE VP4	All Cycles	
NOTE: Only new and unused EZ-TEAR™ Tray Feet Protectors can be sterilized.		

SUPPORTED PRODUCT NUMBERS:

SH-EZT-1		
----------	--	--

ADDITIONAL INFORMATION:

Not made with natural rubber latex.
This is a **single use** product.
Do not store in direct sunlight.



Manufactured By: SURGIHEALTH SOLUTIONS
2122 SCHAPPELLE LN, CINCINNATI, OH 45240
PHONE: 844.88.SURGI | WEB: SURGIHEALTH.COM