

INDIHOLD™

INTENDED USE INFORMATION:

SurgiHealth Solutions INDIHOLD™ is designed to securely hold a variety of chemical indicators, allowing them to remain highly-visible within instrument trays, containers and peel-pouches following sterilization.

APPLICATION:

1. Secure indicator by placing it underneath and through the center band of the IndiHold™ strip.
2. Tuck each end of the indicator under one of the end tabs.
3. Be sure that the indicator is snug between the tabs before placing it in the set.
4. Place the IndiHold™ strip containing the indicator in the tray.

QUALITY INSPECTION:

Ensure that strips are free from tears, debris or contaminants prior to use.

STORAGE:

IndiHold™ strips should be stored in a dry area where cross-contamination can be prevented. Do not store in direct sunlight.

DISPOSAL:

There are no special disposal instructions for this item.

ISSUES:

There are no known product issues or warnings.

STERILIZATION:

Cycle Type	Cycle Temperature	Minimum Cycle Time
Pre Vacuum	134° C (273° F)	3 minutes
Pre Vacuum	132° C (270° F)	4 minutes
Immediate Use Pre Vacuum	132° C (270° F)	3 minutes
Gravity	121° C (250° F)	30 minutes
STERRAD® 100S, 200, 50, NX, 100NX	All Cycles	All Cycles
V-PRO®	All Cycles	
STERIZONE VP4	All Cycles	
NOTE: Only new and unused INDIHOLD™ strips can be sterilized.		

PRODUCT NUMBERS SUPPORTED:

SH-IND-R SH-IND-Y-250	SH-IND-R-250	SH-IND-Y
--------------------------	--------------	----------

ADDITIONAL INFORMATION:

Not made with natural rubber latex.
This is a single use product.
Do not store in direct sunlight.



Manufactured By: SURGIHEALTH SOLUTIONS
2122 SCHAPPELLE LN, CINCINNATI, OH 45240
PHONE: 844.88.SURGI | WEB: SURGIHEALTH.COM