

INSTRUMENT PROTECTION GUARDS

INTENDED USE INFORMATION:

SurgiHealth Solutions Instrument Protection Guards are designed to protect sharp and delicate instrument tips from damage and personnel from injury during handling and reprocessing. The guards are radio opaque and are made from medical-grade polyvinyl chloride (PVC) material.

QUALITY INSPECTION:

Ensure that Instrument Protection Guards are not torn and are free from visible contaminants prior to use.

STORAGE:

Instrument Protection Guard should be stored in a dry area where cross-contamination can be prevented. Do not store in direct sunlight.

DISPOSAL:

There are no special disposal instructions for this item.

ISSUES:

There are no known product issues or warnings.

STERILIZATION:

Cycle Type	Cycle Temperature	Minimum Cycle Time
Pre Vacuum	134° C (273° F)	3 minutes
Pre Vacuum	132° C (270° F)	4 minutes
Immediate Use Pre Vacuum	132° C (270° F)	3 minutes
Gravity	121° C (250° F)	30 minutes

PRODUCT NUMBERS SUPPORTED:

SH-VIG-62-1	SH-VIG-62-2	SH-VIG-78-1
SH-VIG-78-2	SH-VIG-109-1	SH-VIG-109-2
SH-VIG-125-1	SH-VIG-125-2	SH-VIG-187-1
SH-VIG-187-2	SH-VIG-375-1	SH-VIG-375-2
SH-VIG-406-1	SH-VIG-406-2	SH-VIG-625-1
SH-VIG-625-2	SH-VIG-1000-1	SH-VIG-1000-2
SH-DIG-62-2	SH-DIG-78-2	SH-DIG-109-2
SH-DIG-125-2	SH-DIG-187-2	SH-DIG-375-2
SH-GIG-38	SH-GIG-50	SH-GIG-63
SH-WIG-21	SH-WIG-25	

ADDITIONAL INFORMATION:

Not made with natural rubber latex.
This is a single use product.



Manufacturer: SURGIHEALTH SOLUTIONS
2122 SCHAPPELLE LN, CINCINNATI, OH 45240
PHONE: 844.88.SURGI | WEB: SURGIHEALTH.COM