

INSTRUMENT TAPE

INTENDED USE INFORMATION:

SurgiHealth Solutions Instrument Tape is designed to adhere to surgical instruments to help effectively identify, organize and categorize them.

CAUTION: Instrument Tape should not be used on implantable devices.

APPLICATION:

1. Prior to applying instrument tape, clean and dry the surface of the instrument to ensure that the instrument is free of any residue.
2. Wrap instrument tape 1.5 times around the flat and smooth surface of the instrument towards the non-working end of the instrument.
3. Check instrument tape to ensure that there are no gaps between the tape and the instrument surface.
4. Tape will bond to the instrument during the steam sterilization process.

QUALITY INSPECTION:

Replace when tape starts to discolor, chip, crack or flake.

STORAGE:

Instrument tape should be stored in a dry area and where cross-contamination can be prevented.

DISPOSAL:

There are no special disposal instructions for this item.

ISSUES:

There are no known product issues or warnings.

STERILIZATION:

| Cycle Type | Cycle Temperature | Minimum Cycle Time |
|-----------------------------|--------------------|--------------------|
| Pre Vacuum | 134° C (273° F) | 3 minutes |
| Pre Vacuum | 132° C (270° F) | 4 minutes |
| Immediate Use Pre Vacuum | 132° C (270° F) | 3 minutes |
| Gravity | 121° C (250° F) | 30 minutes |

PRODUCT NUMBERS SUPPORTED:

| | | |
|--------------|--------------|--------------|
| SH-STR-18-1 | SH-STR-18-2 | SH-STR-18-3 |
| SH-STR-18-4 | SH-STR-18-5 | SH-STR-18-6 |
| SH-STR-18-7 | SH-STR-18-8 | SH-STR-18-9 |
| SH-STR-14-1 | SH-STR-14-2 | SH-STR-14-3 |
| SH-STR-14-4 | SH-STR-14-5 | SH-STR-14-6 |
| SH-STR-14-7 | SH-STR-14-8 | SH-STR-14-9 |
| SH-SST-14-1 | SH-SST-14-2 | SH-SST-14-3 |
| SH-SST-14-4 | SH-SST-14-5 | SH-SST-14-6 |
| SH-SST-14-7 | SH-SST-14-8 | SH-SST-14-9 |
| SH-SST-14-10 | SH-SST-14-11 | SH-SST-14-12 |
| SH-SST-14-13 | SH-SST-14-14 | SH-SST-14-15 |
| SH-SST-14-16 | SH-SST-14-17 | SH-WST-14-1 |
| SH-WST-14-2 | SH-WST-14-3 | SH-WST-14-4 |
| SH-WST-14-5 | SH-WST-14-6 | SH-WST-14-7 |
| SH-WST-14-8 | SH-WST-14-9 | SH-WST-14-10 |
| SH-WST-14-11 | SH-WST-14-12 | SH-BST-14-1 |
| SH-BST-14-2 | SH-BST-14-3 | SH-BST-14-4 |
| SH-BST-14-5 | SH-BST-14-6 | SH-BST-14-7 |
| SH-BST-14-8 | SH-BST-14-9 | SH-BST-14-10 |
| SH-BST-14-11 | SH-BST-14-1 | |

ADDITIONAL INFORMATION:

Not made with natural rubber latex.
Use by date printed on the package.



Manufactured For: SURGIHEALTH SOLUTIONS
2122 SCHAPPELLE LN, CINCINNATI, OH 45240
PHONE: 844.88.SURGI | WEB: SURGIHEALTH.COM