

MESSAGE TAGS

INTENDED USE INFORMATION:

SurgiHealth Solutions Message Tags are designed to easily communicate information about an instrument, tray and various other equipment.

APPLICATION:

- Using the included lab marker or any marker that has waterproof and alcohol resistant ink, write the information you would like to communicate regarding your item.
- Attach the tag to the item using one of the provided tag ties.

STORAGE:

Tags should be stored in a dry area and where cross-contamination can be prevented. Do not store tags in direct sunlight.

DISPOSAL:

There are no special disposal instructions for this item.

ISSUES:

There are no known product issues or warnings.

STERILIZATION:

Cycle Type	Cycle Temperature	Minimum Cycle Time
Pre Vacuum	134° C (273° F)	3 minutes
Pre Vacuum	132° C (270° F)	4 minutes
Immediate Use Pre Vacuum	132° C (270° F)	3 minutes
Gravity	121° C (250° F)	30 minutes
STERRAD® 100S, 200, 50, NX, 100NX	All Cycles	All Cycles
V-PRO®	All Cycles	
STERIZONE VP4	All Cycles	
NOTE: Only new and unused Message Tags can be sterilized.		

PRODUCT NUMBERS SUPPORTED:

SH-MST-1 SH-MST-4	SH-MST-2 SH-MST-5	SH-MST-3 SH-MST-6
----------------------	----------------------	----------------------

ADDITIONAL INFORMATION:

This is a single use product.



Manufactured For: SURGIHEALTH SOLUTIONS
2122 SCHAPPELLE LN, CINCINNATI, OH 45240
PHONE: 844.88.SURGI | WEB: SURGIHEALTH.COM