



ACETABULAR REAMER BRUSHES - INSTRUCTIONS FOR USE (IFU)

DESCRIPTION:

Brushes designed for manual cleaning of acetabular bone reamers. Constructed with durable handles and nylon bristles for effective debris removal.

INTENDED USE:

For manual pre-cleaning of reusable acetabular bone reamers prior to disinfection and sterilization.

CONTRADICTIONS:

Do not use on instruments that prohibit brush-assisted cleaning. Do not use if bristles are bent, broken, or appear damaged.

WARNINGS AND PRCAUTIONS:

- Single-Use Only. Do not reuse.
- Reuse may result in cross-contamination, infection, or device failure.
- Visually inspect brushes before use.
- Do not use if packaging is damaged or opened.
- Wear personal protective equipment (PPE) during use.
- Ensure brush size is appropriate for channel diameter.

INDICATIONS FOR USE:

Use during the manual cleaning stage to remove visible soil and debris from acetabular reamer surfaces and internal cavities.

DIRECTIONS FOR USE:

1. Confirm correct brush size for the reamer being cleaned.
2. Open the sterile or clean package in a controlled environment.
3. Rinse the brush under cool water to remove loose debris.
4. Apply enzymatic solution according to manufacturer instructions and scrub reamer surfaces.
5. Rinse thoroughly and inspect to ensure all visible soil is removed.
6. After use, discard brush immediately in a biohazard container.
7. Do not attempt to clean, disinfect, or sterilize this brush.

STORAGE CONDITIONS:

Store in a clean, dry environment at room temperature. Avoid excessive heat, humidity, and direct sunlight.


SHELF LIFE AND DISPOSAL

- Shelf life: 5 years from date of manufacture.
- Dispose of used brushes according to local medical waste disposal regulations.

SUPPORTED PRODUCT SKUS:

SHACE1		
--------	--	--

NOTE: This IFU applies to all listed product SKUs regardless of packaging configuration.



Manufactured For: SURGIHEALTH SOLUTIONS
2122 SCHAPPELLE LN, CINCINNATI, OH 45240
PHONE: 844.88.SURGI | WEB: SURGIHEALTH.COM