



CHANNEL CLEANING BRUSHES - INSTRUCTIONS FOR USE (IFU)

DESCRIPTION:

Sterile processing brushes designed for manual cleaning of lumened surgical instruments and suction channels. Constructed with twisted stainless-steel wire, nylon bristles, and a loop or ring handle for safe handling.

INTENDED USE:

For manual pre-cleaning of reusable surgical instruments, specifically lumens, channels, and suction tubes, prior to disinfection and sterilization.

CONTRADICTIONS:

Do not use on instruments that prohibit brush-assisted cleaning. Do not use if bristles are bent, broken, or appear damaged.

WARNINGS AND PRCAUTIONS:

- Single-Use Only. Do not reuse.
- Reuse may result in cross-contamination, infection, or device failure.
- Visually inspect brushes before use.
- Do not use if packaging is damaged or opened.
- Wear personal protective equipment (PPE) during use.
- Ensure brush size is appropriate for channel diameter.

INDICATIONS FOR USE:

Use during the manual cleaning stage to remove visible soil and debris from internal channels of surgical instruments.

DIRECTIONS FOR USE:

1. Confirm correct brush size matches the instrument channel diameter.
2. Open the sterile or clean package in a controlled environment.
3. Insert the brush gently into the channel and move back and forth to remove debris.
4. Repeat as needed until all visible soil is removed.
5. After use, discard brush immediately in a biohazard container.
6. Do not attempt to clean, disinfect, or sterilize this brush.

STORAGE CONDITIONS:

Store in a clean, dry environment at room temperature. Avoid excessive heat, humidity, and direct sunlight.


SHELF LIFE AND DISPOSAL

- Shelf life: 5 years from date of manufacture.
- Dispose of used brushes according to local medical waste disposal regulations.

SUPPORTED PRODUCT SKUS:

SHTSB07039	SHTSB16039	SHTSB11067
SHTSB16079	SHTSB09079	SHTSB12079
SHTSB09091	SHTSB24098	SHTSB09106
SHTSB09118	SHTSB12118	SHTSB16118
SHTSB09130	SHTSB09146	SHTSB09157
SHTSB12157	SHTSB16157	SHTSB08197
SHTSB12197	SHTSB16197	SHTSB24197
SHTSB08236	SHTSB12236	SHTSB16236
SHTSB24236	SHTSB08276	SHTSB16276
SHTSB24276	SHTSB08315	SHTSB24315
SHTSB08393	SHTSB12393	SHTSB16393
SHTSB24393	SHTSB08433	SHTSB16433
SHTSB08472	SHTSB16472	SHTSB24472
SHTSB24591	SHTSB08630	SHTSB16591
SHTSB16787	SHTSB24787	SHTSB08787
SHTSB08944	SHTSB16944	SHTSB16886

NOTE: ThisIFU applies to all listed SKUs regardless of packaging configuration.



Manufactured For: SURGIHEALTH SOLUTIONS
2122 SCHAPPELLE LN, CINCINNATI, OH 45240
PHONE: 844.88.SURGI | WEB: SURGIHEALTH.COM