



FACE SHIELDS - INSTRUCTIONS FOR USE (IFU)

DESCRIPTION:

Disposable face shields designed to provide barrier protection for the eyes, nose, and mouth against fluid splashes and airborne particles. Lightweight, anti-fog, and made with a soft foam headband and adjustable strap.

INTENDED USE:

For single-use barrier protection in clinical, sterile processing, and other healthcare environments where facial protection is required.

CONTRADICTIONS:

Not intended for use in surgical procedures, high-heat environments, or as a replacement for respirators. Do not reuse or share between users.

WARNINGS AND PRCAUTIONS:

- **Single-Use Only. Do not reuse.**
- Do not autoclave, disinfect, or sterilize.
- Inspect shield and headband before use. Discard if damaged or contaminated.
- Use in combination with other PPE as required by facility protocol.

INDICATIONS FOR USE:

Wear over the face to protect against fluid splashes, debris, and respiratory droplets during cleaning, processing, or other healthcare tasks.

DIRECTIONS FOR USE:

1. Remove shield from packaging with clean hands or gloves.
2. Peel off protective film from the shield surface (if present).
3. Place foam headband against forehead and adjust strap around the head until secure.
4. Confirm full coverage of face and a clear line of vision.
5. Discard after use. Do not attempt to clean or disinfect.

STORAGE CONDITIONS:

Store flat in a clean, dry environment at room temperature. Avoid direct sunlight, high humidity, or crushing weight.

SHELF LIFE AND DISPOSAL

- Shelf Life: Indefinite if stored under ideal storage conditions.
- Disposal: Discard after use with general PPE waste, following facility guidelines.

SUPPORTED PRODUCT SKUS:

SHFCS1		
--------	--	--

NOTE: This IFU applies to all listed product SKUs regardless of packaging configuration.



Manufactured By: SURGIHEALTH SOLUTIONS
2122 SCHAPPELLE LN, CINCINNATI, OH 45240
PHONE: 844.88.SURGI | WEB: SURGIHEALTH.COM