



INDIHOLD™ - INSTRUCTIONS FOR USE (IFU)

DESCRIPTION:

Single-use retention strips designed to securely hold chemical indicators inside instrument trays, containers, or peel pouches for easy visibility post-sterilization.

INTENDED USE:

For holding chemical indicator strips securely in place during sterilization to ensure post-sterilization visibility.

CONTRADICTIONS:

Do not use if product appears damaged or compromised. Do not reuse. Only use with compatible chemical indicators.

WARNINGS AND PRCAUTIONS:

- **Single-Use Only. Do not reuse.**
- Reuse may result in contamination or failed sterilization monitoring.
- Visually inspect strips before use.
- Do not use if packaging is damaged or opened.
- Store away from direct sunlight and moisture.

INDICATIONS FOR USE:

Use in trays, containers, or pouches to retain indicator strips in a visible and secure location throughout the sterilization cycle.

DIRECTIONS FOR USE:

1. Insert the chemical indicator under the center band of the IndiHold™ strip.
2. Tuck each end of the indicator under the end tabs.
3. Ensure the indicator is snug and secure before placing in the tray or container.
4. Discard strip after use. Do not reuse or attempt to clean.

STORAGE CONDITIONS:

Store in a clean, dry environment at room temperature. Avoid excessive heat, humidity, and direct sunlight.

SHELF LIFE AND DISPOSAL

- Shelf life: 5 years from date of manufacture.
- Dispose of used brushes according to local medical waste disposal regulations.


STERILIZATION COMPATIBILITY:

INDIHOLD™ is compatible with all standard sterilization methods, including steam, STERRAD®, V-PRO®, and STERIZONE® systems. Only new and unused strips should be sterilized.

SUPPORTED PRODUCT SKUS:

SHINDR	SHINDY	
--------	--------	--

NOTE: This IFU applies to all listed product SKUs regardless of packaging configuration.



Manufactured By: SURGIHEALTH SOLUTIONS
2122 SCHAPPELLE LN, CINCINNATI, OH 45240
PHONE: 844.88.SURGI | WEB: SURGIHEALTH.COM

INDIHOLD

