



WIDE LOOP TAGS + INSTRUMENT LOOP TAGS - INSTRUCTIONS FOR USE (IFU)

DESCRIPTION:

Durable single-use tags made from sterilization-compatible material, designed to convey key status messages, warnings, or instructions when affixed to instruments, trays, or bins.

INTENDED USE:

For clearly displaying important visual cues on instruments or containers during sterilization, storage, or transport. Compatible with all standard sterilization methods.

CONTRADICTIONS:

Do not use if tag appears torn or degraded.

WARNINGS AND PRCAUTIONS:

- **Single-Use Only. Do not reuse.**
- Visually inspect before use.
- Do not apply damaged or contaminated tags.
- Follow facility protocols for interpreting message tag meanings.

INDICATIONS FOR USE:

Apply to trays, bins, or instruments to indicate specific instructions (e.g., “Do Not Use,” “Needs Repair,” “Immediate Use Only”).

DIRECTIONS FOR USE:

1. Fill in relevant information (optional).
2. Loop securely through instrument, tray handle, or tray slot.
3. Ensure tag is clearly visible and secure.
4. Discard after use. Do not reuse.

STORAGE CONDITIONS:

Store in a clean, dry environment at room temperature. Avoid direct sunlight and excess humidity.

SHELF LIFE AND DISPOSAL

- Shelf Life: Indefinite if stored under ideal storage conditions (see above).
- Disposal: Discard used indicators with general medical waste per facility policy.


STERILIZATION COMPATIBILITY:

Message Tags are compatible with all standard sterilization methods, including steam, STERRAD®, V-PRO®, and STERIZONE® systems. Only new and unused tags should be sterilized.

SUPPORTED PRODUCT SKUS:

SHWLT1	SHWLT2	SHWLT3
SHWLT4	SHWLT5	SHWLT6
SHWLT7		
-----	-----	-----
SHILT1	SHILT2	SHILT3
SHILT4	SHILT5	SHILT6
SHILT7		

NOTE: This IFU applies to all listed product SKUs regardless of packaging configuration.



Manufactured By: SURGIHEALTH SOLUTIONS
2122 SCHAPPELLE LN, CINCINNATI, OH 45240
PHONE: 844.88.SURGI | WEB: SURGIHEALTH.COM