



PLASTIC HANDLE BRUSHES - INSTRUCTIONS FOR USE (IFU)

DESCRIPTION:

Single-use cleaning brushes with ergonomic plastic handles and durable nylon bristles. Designed to provide comfortable, effective manual cleaning of surgical instruments and medical surfaces.

INTENDED USE:

For manual cleaning of reusable surgical instruments during the pre-cleaning and decontamination process.

CONTRADICTIONS:

Do not use on instruments that prohibit brush-assisted cleaning. Do not use if bristles are bent, broken, or brush appears damaged.

WARNINGS AND PRECAUTIONS:

- **Single-Use Only. Do not reuse.**
- Do not attempt to clean, disinfect, or sterilize.
- Always inspect brush prior to use.
- Wear appropriate personal protective equipment (PPE).
- Use only with compatible enzymatic or detergent solutions.

INDICATIONS FOR USE:

Use to remove visible soil, debris, and organic matter from surgical instruments before high-level disinfection or sterilization.

DIRECTIONS FOR USE:

1. Select the appropriate brush size and bristle stiffness for the task.
2. Rinse brush under cool water to remove gross debris.
3. Apply enzymatic solution per manufacturer’s instructions.
4. Scrub instrument surfaces thoroughly, including hinges, serrations, and lumens where applicable.
5. Rinse and visually inspect the instrument and brush.
6. Discard brush immediately after use in a biohazard container.

STORAGE CONDITIONS:

Store in a clean, dry area away from sunlight, extreme temperatures, or contaminants.

SHELF LIFE AND DISPOSAL

- Shelf Life: Indefinite if stored under ideal storage conditions (see above).
- Disposal: Discard used indicators with general medical waste per facility policy.

STERILIZATION COMPATIBILITY:

Do not sterilize. Plastic Handle Brushes are designed for single-use and should not be included in any sterilization cycle.


STERILIZATION COMPATIBILITY:

Do not sterilize. Plastic Handle Brushes are designed for single-use and should not be included in any sterilization cycle.

SUPPORTED PRODUCT SKUS:

SHPHB1	SHPHB2	SHPHB3
SHPHB4	SHPHB5	SHPHB6
SHPHB7	SHPHB8	SHPHB9

NOTE: This IFU applies to all listed product SKUs regardless of packaging configuration.



Manufactured For: SURGIHEALTH SOLUTIONS
2122 SCHAPPELLE LN, CINCINNATI, OH 45240
PHONE: 844.88.SURGI | WEB: SURGIHEALTH.COM