

## DISPOSABLE CORNER PROTECTORS (FOAM)

### INTENDED USE INFORMATION:

SurgiHealth Solutions Disposable Corner Protectors are made of foam and are designed to prevent tears and punctures in surgical tray wraps associated with sharp corners and feet on surgical trays.

### APPLICATION:

1. Separate corner protectors into four individual corners.
2. Place corner protectors on the top and bottom corners of the tray prior to wrapping.

### QUALITY INSPECTION:

Ensure that corner protectors are free from debris or contaminants prior to use.

### STORAGE:

Corner protectors should be stored in a dry area and where cross-contamination can be prevented.

### DISPOSAL:

There are no special disposal instructions for this item.

### ISSUES:

There are no known product issues or warnings.

### STERILIZATION:

| Cycle Type | Cycle Temperature  | Minimum Cycle Time |
|------------|--------------------|--------------------|
| Pre Vacuum | 132° C<br>(270° F) | 4 minutes          |
| Gravity    | 121° C<br>(250° F) | 30 minutes         |

**NOTE:** Only new and unused corner protectors can be sterilized.

### SUPPORTED PRODUCT NUMBERS:

SH-TCP-3

### ADDITIONAL INFORMATION:

This is a [single use](#) product.



**Manufactured For:** SURGIHEALTH SOLUTIONS  
2122 SCHAPPELLE LN, CINCINNATI, OH 45240  
PHONE: 844.88.SURGI | WEB: SURGIHEALTH.COM