

## DISPOSABLE CORNER PROTECTORS (SILICONE)

### INTENDED USE INFORMATION:

SurgiHealth Solutions Disposable Corner Protectors are made of medical-grade silicone and are designed to prevent tears and punctures in surgical tray wraps associated with sharp corners and feet on surgical trays.

### APPLICATION:

1. Place corner protectors on the top and bottom corners of the tray prior to wrapping.

### QUALITY INSPECTION:

Ensure that corner protectors are free from debris or contaminants prior to use.

### STORAGE:

Corner protectors should be stored in a dry area and where cross-contamination can be prevented.

### DISPOSAL:

There are no special disposal instructions for this item.

### ISSUES:

There are no known product issues or warnings.

### STERILIZATION:

Cycle Type	Cycle Temperature	Minimum Cycle Time
Pre Vacuum	134° C (273° F)	3 minutes
Pre Vacuum	132° C (270° F)	4 minutes
Immediate Use Pre Vacuum	132° C (270° F)	3 minutes
Gravity	121° C (250° F)	30 minutes
STERRAD® 100S, 200, 50, NX, 100NX	All Cycles	All Cycles
<b>NOTE:</b> Only new and unused corner protectors can be sterilized.		

### SUPPORTED PRODUCT NUMBERS:

SH-TCP-2

### ADDITIONAL INFORMATION:

Not made with natural rubber latex.  
This is a single use product.



**Manufactured For:** SURGIHEALTH SOLUTIONS  
2122 SCHAPPELLE LN, CINCINNATI, OH 45240  
PHONE: 844.88.SURGI | WEB: SURGIHEALTH.COM