

## DRY ENDOSCOPE SPONGES

### INTENDED USE INFORMATION:

SurgiHealth Solutions Dry Endoscope Sponges are designed to remove gross soil and debris from endoscopes during decontamination.

**CAUTION:** Only new and unopened sponges should be used on soiled endoscopes. Discard sponges immediately after use. SurgiHealth Solutions Dry Endoscope Sponges are non-sterile and single use.

### APPLICATION:

- Sponges can be used dry or with enzymatic solution.
  

  1. Position sponge around endoscope tube.
  2. Gently wipe down the length of the endoscope tube to remove gross soil and debris.
  3. Discard sponge immediately after use.

### STORAGE:

Brushes should be stored in a dry area and where cross-contamination can be prevented.

### DISPOSAL:

There are no special disposal instructions for this item.

### ISSUES:

There are no known product issues or warnings.

### PRODUCT NUMBERS SUPPORTED:

SH-DES-1		
----------	--	--

### ADDITIONAL INFORMATION:

This is a single use product.



**Manufactured For:** SURGIHEALTH SOLUTIONS  
2122 SCHAPPELLE LN, CINCINNATI, OH 45240  
PHONE: 844.88.SURGI | WEB: SURGIHEALTH.COM