

EASY-BREAK SEALS

INTENDED USE INFORMATION:

SurgiHealth Solutions Easy-Break Seals are ideal for grouping and identifying instruments, instrument trays, loaner instrumentation and various other equipment.

QUALITY INSPECTION:

Ensure that seals are not broken and in working order prior to use.

STORAGE:

Easy-Break Seals should be stored in a dry area and where cross-contamination can be prevented. Do not store seals in direct sunlight.

DISPOSAL:

There are no special disposal instructions for this item.

ISSUES:

There are no known product issues or warnings.

STERILIZATION:

Cycle Type	Cycle Temperature	Minimum Cycle Time
Pre Vacuum	134° C (273° F)	3 minutes
Pre Vacuum	132° C (270° F)	4 minutes
Immediate Use Pre Vacuum	132° C (270° F)	3 minutes
Gravity	121° C (250° F)	30 minutes
STERRAD® 100S, 200, 50, NX, 100NX	All Cycles	All Cycles

PRODUCT NUMBERS SUPPORTED:

SH-EBS-1N SH-EBS-4N SH-EBS-7N	SH-EBS-2N SH-EBS-5N	SH-EBS-3N SH-EBS-6N
-------------------------------------	------------------------	------------------------

ADDITIONAL INFORMATION:

This is a [single use](#) product.



Manufactured For: SURGIHEALTH SOLUTIONS
2122 SCHAPPELLE LN, CINCINNATI, OH 45240
PHONE: 844.88.SURGI | WEB: SURGIHEALTH.COM