

# ENDOSCOPE BRUSHES

## INTENDED USE INFORMATION:

SurgiHealth Solutions Endoscope Brushes are designed to clean the channels of endoscopes and channeled instruments during the cleaning and decontamination processes. The brushes fill material consist of nylon.

**CAUTION:** Only new and unopened brushes should be used on soiled endoscopes. Discard brushes immediately after use. SurgiHealth Solutions Endoscope Brushes are non-sterile and single use.

## QUALITY INSPECTION:

Ensure that brushes are free from debris or contaminants and not chipped, bent or damaged prior to use. Please discard and replace brushes after each use. Continued use of brushes can result in cross contamination, and may cause damage to the medical device .

## STORAGE:

Brushes should be stored in a dry area and where cross-contamination can be prevented.

## DISPOSAL:

There are no special disposal instructions for this item.

## ISSUES:

There are no known product issues or warnings.

## STERILIZATION:

Cycle Type	Cycle Temperature	Minimum Cycle Time
Pre Vacuum	134° C (273° F)	3 minutes
Pre Vacuum	132° C (270° F)	4 minutes
Immediate Use Pre Vacuum	132° C (270° F)	3 minutes
Gravity	121° C (250° F)	30 minutes

**NOTE:** Only new and unused brushes can be sterilized. Brushes may also show slight discoloration due to the sterilization process.

## PRODUCT NUMBERS SUPPORTED:

SH-ESB-94-079 SH-ESB-94-197	SH-ESB-94-118 SH-ESB-59-610	SH-ESB-94-157 SH-ESB-45-393
--------------------------------	--------------------------------	--------------------------------

**NOTE:** Includes 1/PK and 50/PK quantities.

### ADDITIONAL INFORMATION:

Not made with natural rubber latex.  
This is a single use product.



**Manufactured For:** SURGIHEALTH SOLUTIONS  
2122 SCHAPPELLE LN, CINCINNATI, OH 45240  
PHONE: 844.88.SURGI | WEB: SURGIHEALTH.COM