

FACE SHIELDS

INTENDED USE INFORMATION:

SurgiHealth Solutions are designed to protect against liquid splashes and sprays.

APPLICATION: (Non Adjustable)

1. Remove shield from packaging.
2. Place the elastic band over top of head with foam facing to the front.
3. Adjust for comfort.

APPLICATION: (Adjustable)

1. Remove shield from packaging.
2. Place the elastic band over top of head with foam facing to the front.
3. Remove shield to adjust elastic band using slots to loosen or tighten to desired fit
4. Adjust for comfort.

DISPOSAL:

There are no special disposal instructions for this item.

ISSUES:

There are no known product issues or warnings.

PRODUCT NUMBERS SUPPORTED:

SH-FCS-1	SH-FCS-2	
----------	----------	--

ADDITIONAL INFORMATION:

This is a single use product.



Manufactured For: SURGIHEALTH SOLUTIONS
2122 SCHAPPELLE LN, CINCINNATI, OH 45240
PHONE: 844.88.SURGI | WEB: SURGIHEALTH.COM