

# INSTRUMENT COUNTER + ORGANIZER

## INTENDED USE INFORMATION:

SurgiHealth Solutions Instrument Counter + Organizers are designed to hold a variety of instruments for counting and organization before, during and following a procedure. In addition to keeping sharp tips pointing down and secure.

## APPLICATION:

1. Place Instrument Counter + Organizer on a flat surface.
2. Lay up to 16 individual instruments (ring handled) in the numbered slots with the sharp tips pointing down.

**NOTE:** Connect two (2) or more Instrument Counter + Organizers together by inserting the two (2) pegs into the two slots on the opposite end of each additional Instrument Counter + Organizer.

## QUALITY INSPECTION:

Ensure that Instrument Counter + Organizers are free from cracks, chips, debris or contaminants prior to use.

## STORAGE:

Store in a dry area and where cross-contamination can be prevented. Do not store in direct sunlight.

## DISPOSAL:

There are no special disposal instructions for this item.

## ISSUES:

There are no known product issues or warnings.

## STERILIZATION:

Cycle Type	Cycle Temperature	Minimum Cycle Time
Pre Vacuum	134° C (273° F)	3 minutes
Pre Vacuum	132° C (270° F)	4 minutes

## PRODUCT NUMBERS SUPPORTED:

SH-ICO-1		
----------	--	--

**ADDITIONAL INFORMATION:**  
This is a [single use](#) product.  
Do not store in direct sunlight.



**Manufactured For:** SURGIHEALTH SOLUTIONS  
2122 SCHAPPELLE LN, CINCINNATI, OH 45240  
PHONE: 844.88.SURGI | WEB: SURGIHEALTH.COM