

# INSTRUMENT PROTECTION

**INTENDED USE INFORMATION:**

SurgiHealth Solutions Instrument Protection guards are designed to protect sharp and delicate instrument tips from damage and personnel from injury during handling and reprocessing. The guards are radio opaque and are made from medical-grade polyvinyl chloride (PVC) material.

**QUALITY INSPECTION:**

Ensure that Instrument Protection guards are not torn and are free from visible contaminants prior to use.

**STORAGE:**

Instrument Protection guards should be stored in a dry area and where cross-contamination can be prevented. Do not store guards in direct sunlight.

**DISPOSAL:**

There are no special disposal instructions for this item.

**ISSUES:**

There are no known product issues or warnings.

**STERILIZATION:**

Cycle Type	Cycle Temperature	Minimum Cycle Time
Pre Vacuum	134° C (273° F)	3 minutes
Pre Vacuum	132° C (270° F)	4 minutes
Immediate Use Pre Vacuum	132° C (270° F)	3 minutes
Gravity	121° C (250° F)	30 minutes

**PRODUCT NUMBERS SUPPORTED:**

SH-VIG-62-1	SH-VIG-62-2	SH-VIG-62-3
SH-VIG-78-1	SH-VIG-78-2	SH-VIG-78-3
SH-VIG-109-1	SH-VIG-109-2	SH-VIG-109-3
SH-VIG-125-1	SH-VIG-125-2	SH-VIG-125-3
SH-VIG-187-1	SH-VIG-187-2	SH-VIG-187-3
SH-VIG-406-1	SH-VIG-406-2	SH-VIG-406-3
SH-VIG-375-1	SH-VIG-375-2	SH-VIG-375-3
SH-VIG-625-1	SH-VIG-625-2	SH-VIG-625-3
SH-VIG-1000-1	SH-VIG-1000-2	SH-VIG-1000-3
SH-DIG-62-2	SH-DIG-78-2	SH-DIG-109-2
SH-DIG-125-2	SH-DIG-187-2	SH-DIG-375-2
SH-OIG-250	SH-OIG-375	SH-OIG-500
SH-OIG-750	SH-OIG-1250	SH-OIG-1500
SH-OIG-1750	SH-GIG-38	SH-GIG-50
SH-GIG-63	SH-WIG-20	SH-WIG-25

**ADDITIONAL INFORMATION:**

Not made with natural rubber latex.  
This is a single use product.



**Manufactured For:** SURGIHEALTH SOLUTIONS  
2122 SCHAPPELLE LN, CINCINNATI, OH 45240  
PHONE: 844.88.SURGI | WEB: SURGIHEALTH.COM