

## INSTRUMENT SLEEVES

### INTENDED USE INFORMATION:

SurgiHealth Solutions Instrument Sleeves are designed to protect sharp and delicate instrument tips from damage and personnel from injury during handling. Instrument Sleeves are tear-resistant and compatible with steam and gas plasma sterilization.

### QUALITY INSPECTION:

Ensure that Instrument Sleeves are not torn and are free from visible contaminants prior to use.

### STORAGE:

Instrument Sleeves should be stored in a dry area and where cross-contamination can be prevented.

### DISPOSAL:

There are no special disposal instructions for this item.

### ISSUES:

There are no known product issues or warnings.

### STERILIZATION:

Cycle Type	Cycle Temperature	Minimum Cycle Time
Pre Vacuum	134° C (273° F)	3 minutes
Pre Vacuum	132° C (270° F)	4 minutes
Immediate Use Pre Vacuum	132° C (270° F)	3 minutes
Gravity	121° C (250° F)	30 minutes
STERRAD® 100S, 200, 50, NX, 100NX	All Cycles	All Cycles
V-PRO®	All Cycles	
STERIZONE VP4	All Cycles	
<b>NOTE:</b> Only new and unused Instrument Sleeves can be sterilized.		

### PRODUCT NUMBERS SUPPORTED:

SH-ISL-3	SH-ISL-6	
----------	----------	--

### ADDITIONAL INFORMATION:

Not made with natural rubber latex.  
This is a single use product.



**Manufacturer:** SURGIHEALTH SOLUTIONS  
2122 SCHAPPELLE LN, CINCINNATI, OH 45240  
PHONE: 844.88.SURGI | WEB: SURGIHEALTH.COM