

# LOCKING TAGS

**INTENDED USE INFORMATION:**

SurgiHealth Solutions Locking Tags are ideal for grouping and identifying instruments, instrument trays, loaner instrumentation and various other equipment.

**QUALITY INSPECTION:**

Ensure that tags are not broken and in working order prior to use.

**STORAGE:**

Locking Tags should be stored in a dry area and where cross-contamination can be prevented. Do not store tags in direct sunlight.

**DISPOSAL:**

There are no special disposal instructions for this item.

**ISSUES:**

There are no known product issues or warnings.

**STERILIZATION:**

Cycle Type	Cycle Temperature	Minimum Cycle Time
Pre Vacuum	134° C (273° F)	3 minutes
Pre Vacuum	132° C (270° F)	4 minutes
Immediate Use Pre Vacuum	132° C (270° F)	3 minutes
Gravity	121° C (250° F)	30 minutes
STERRAD® 100S, 200, 50, NX, 100NX	All Cycles	All Cycles

**PRODUCT NUMBERS SUPPORTED:**

SH-LOT-1	SH-LOT-2	SH-LOT-3
SH-LOT-4	SH-LOT-5	SH-LOT-6
SH-LOT-7	SH-LOT-1N	SH-LOT-2N
SH-LOT-3N	SH-LOT-4N	SH-LOT-5N
SH-LOT-6N	SH-LOT-7N	SH-LOT-1P
SH-LOT-2P	SH-LOT-3P	SH-LOT-4P
SH-LOT-5P	SH-LOT-6P	SH-LOT-7P

**ADDITIONAL INFORMATION:**

This is a [single use](#) product.



**Manufactured For:** SURGIHEALTH SOLUTIONS  
 2122 SCHAPPELLE LN, CINCINNATI, OH 45240  
 PHONE: 844.88.SURGI | WEB: SURGIHEALTH.COM