

MAGNETIC INSTRUMENT MATS (SINGLE USE)

INTENDED USE INFORMATION:

SurgiHealth Solutions Magnetic Instrument Mats are designed to provide a safe way to transfer sharp surgical instruments with the sterile field during procedures.

CAUTION: Do not use magnetic instrument mats in an MRI setting.

Only new and unused magnetic instrument mats should be sterilized for use. Discard mats immediately after use. SurgiHealth Solutions magnetic instrument mats are non-sterile and single use.

QUALITY INSPECTION:

Ensure that mats are free from tears, debris or contaminants prior to use.

STORAGE:

Magnetic instrument mats should be stored in a dry area and where cross-contamination can be prevented. Do not store in direct sunlight.

DISPOSAL:

There are no special disposal instructions for this item.

ISSUES:

There are no known product issues or warnings.

STERILIZATION:

Cycle Type	Cycle Temperature	Minimum Cycle Time
Pre Vacuum	134° C (273° F)	3 minutes
Pre Vacuum	132° C (270° F)	4 minutes
Gravity	121° C (250° F)	30 minutes

NOTE: Only new and unused magnetic instrument mats can be sterilized.

PRODUCT NUMBERS SUPPORTED:

SH-MIM-1	SH-MIM-2	
----------	----------	--

ADDITIONAL INFORMATION:

Not made with natural rubber latex.
This is a single use product.
Do not store in direct sunlight.



Manufactured For: SURGIHEALTH SOLUTIONS
2122 SCHAPPELLE LN, CINCINNATI, OH 45240
PHONE: 844.88.SURGI | WEB: SURGIHEALTH.COM