

TRAY CORNER PROTECTORS (FOAM)

INTENDED USE INFORMATION:

SurgiHealth Solutions Disposable Corner Protectors are made of foam and are designed to prevent tears and punctures in surgical tray wraps associated with sharp corners and feet on surgical trays.

APPLICATION:

1. Separate corner protectors into four individual corners.
2. Place corner protectors on the top and bottom corners of the tray prior to wrapping.

QUALITY INSPECTION:

Ensure that corner protectors are free from debris or contaminants prior to use.

STORAGE:

Corner protectors should be stored in a dry area and where cross-contamination can be prevented.

DISPOSAL:

There are no special disposal instructions for this item.

ISSUES:

There are no known product issues or warnings.

STERILIZATION:

Cycle Type	Cycle Temperature	Minimum Cycle Time
Pre Vacuum	132° C (270° F)	4 minutes
Gravity	121° C (250° F)	30 minutes

NOTE: Only new and unused corner protectors can be sterilized.

SUPPORTED PRODUCT NUMBERS:

SH-TCP-3

ADDITIONAL INFORMATION:

This is a [single use](#) product.



Manufactured For: SURGIHEALTH SOLUTIONS
2122 SCHAPPELLE LN, CINCINNATI, OH 45240
PHONE: 844.88.SURGI | WEB: SURGIHEALTH.COM